Somatic cell test for subclinical mastitis

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EN The PortaSCC® milk test can be used to:

- Identify problem cows or quarters
- Monitor response to treatment
- Check cows at freshening and dry off
- Monitor udder health
- Screen a herd or a group of cows

Precautions:

- Wash and dry sample containers thoroughly before use.
- Preservatives such as Bronopol will prevent the test from working.
- Antibiotics will not interfere with the test.
- Milk samples must be tested within 8 hours of milking.
- If milk is refrigerated, allow it to warm to room temperature before testing.
- Run the test in a shaded area out of direct sunlight at temperatures between 7° C and 35° C (45° F and 95° F).
- Mix the sample before testing.
- Use only the pipette provided in the kit.
- Do not touch the pipet tip to the test strip when applying sample.

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ES Contenido

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FR Contenu

Test Strips - Strips are sensitive to humidity. Use within 3 months of opening vial.

ATTENTION: Close vial immediately after removing a test strip.

Activator Solution - Contains preservative. Do not drink.

Color Chart - For estimating the somatic cell count.

Pipettes, reusable - To use the pipette, insert tip into milk, squeeze and slowly release bulb. Gently squeeze the bulb to dispense one drop of milk. To reuse, discard remaining milk and rinse the pipette in the next sample or in water by squeezing and releasing the bulb 3-4 times.

Blanking Strip(s), reusable For use with Digital Reader only (Digital Reader sold separately). Store in resealable bag when not in use.

Storage: Store the kit at 2° C - 25° C (36° F - 77° F). Refrigerate if possible. Use by the "Best if used by" date on the label.

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EN Intended Use

ES Uso Previsto

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This test is intended solely for the estimation of somatic cell count in fresh cow milk. The test is not a laboratory reference method and should not be used as a diagnostic test. Consult a veterinarian before starting any treatment.

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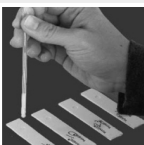
EN Running the Test

Test Strips can be run individually or in groups. Write sample ID number on each test strip and sample container before testing.

- Forestrip
- Collect a milk sample into a clean labeled container. The container does not have to be sterile. Mix the sample.

- Add 1 drop of milk to the test strip sample well using the pipette provided. Let the milk absorb into the well.

- Add 3 drops of Activator Solution to the sample well.

Quick screen:

- If color starts to develop in a few minutes, the cell count is very high.

- Wait 45 minutes before using the Color Chart or Digital Reader.

After 45 minutes

- Estimate the somatic cell count by comparing the strip to the Color Chart or by using the Digital Reader.

For more detailed Digital Reader instructions, refer to the User Guide included with the Digital Reader.

Principle of the Test:

The PortaSCC® milk test was developed for estimating the somatic cell count in individual cows. The test has been evaluated at universities in the USA and Canada and has been shown to have a good correlation with laboratory results.

The PortaSCC® milk test is based on a chemical reaction between a dye on the test strip and an enzyme found on the cells in the milk. This reaction makes the test strip sample well change to a blue color. The darker the blue color, the higher the cell count.

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